

ECG 247 Heart Sensor

USER GUIDE
MODEL 353 010

1

Download and install the ECG247 app on your mobile phone



2

Click the link in received SMS from your physician or open the app and follow the instructions



3

Enter your personal information in the app



4

Place the sensor on the electrode




Do not remove the green foil over the battery

Rotate clockwise

5

Connect the sensor to the phone

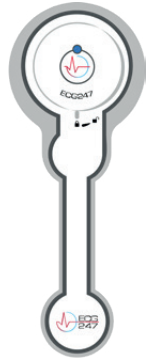

Press the center of the sensor until the blue light comes on

6


Clean the skin



Wash and dry the skin.
Hair must be removed

7

Attach the sensor on the upper part of sternum

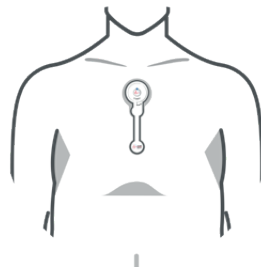

Remove the protective paper

8

The heart rhythm is recorded and analyzed



You can now use the phone as usual, but be sure to keep your phone in proximity during the test period

9

Use the sensor for at least 3 days. The test is stopped by pressing «Stop test»

Stop test

10

Order a review of the test results from our heart specialists or share the results with your physician



Order new sensors and electrodes at
ECG247.com

PRIVACY Your privacy is important to us. We protect your data and do not share your information. Read more at ecg247.com

RESPONSIBILITY The heart signals have different shapes and sizes in different people. We cannot guarantee that this test finds all different rhythm disorders in everyone. The sensor cannot look for signs of myocardial infarction. Any rhythm disturbances must always be confirmed by a doctor. Read more at ecg247.com

Always contact a physician if you have symptoms with your heart.

WARNING The sensor should not be used in children, pregnant women, significantly overweight (BMI>40), patients with pacemaker/ICD and in situations requiring external defibrillator use. If the use causes itching or rash, the sensor should be removed. Do not swallow. The sensor must be kept out of reach of children. The battery cannot be charged.

ENVIRONMENT Remove battery from the electrode for proper recycling.

You can shower with the sensor, but avoid bathing.

Complete instructions at ECG247.com



Made in Norway
AppsSens AS
Senterveien 30
4790 Lillesand, Norway
www.ecg247.com

Art. 23186 / Rev. 02/2010